ENG The fast and safe way to evacuate bed-ridden people. Can be easily dragged by just one rescuer.

Safety information

These instructions contain information concerning the safe use of the S-CAPEPOD. To ensure that the S-CAPEPOD is used correctly and safely, this information must be read prior to placement and usage of the S-CAPEPOD.

- Always observe the instructions given in the instruction manual when using the S-CAPEPOD.
- Only use the S-CAPEPOD in emergencies, such as a fire, power cut, gas leak, terrorist attack, natural disaster, extreme smoke or any other life-threatening situation or emergency.
- Product training with the S-CAPEPOD must always be given by a certified emergency response team member.
- Incorrect placement of the S-CAPEPOD between the bed frame and the mattress may have a negative effect on the effectiveness of the S-CAPEPOD. Therefore, always strictly observe the instructions given in the fold-out leaflet.
- The use of alcohol and/or prescription and over-the-counter medicine may have a negative effect on the skill of the rescuer who pulls the S-CAPEPOD.
- Do not remove any parts of the S-CAPEPOD.
- Make sure the safety information and instruction manual are kept close to the bed where an S-CAPEPOD is located.
- This S-CAPEPOD is not intended for recreational use.
- Do not use flammable vapours, materials or liquids, such as petrol, alcohol, etc., close to the S-CAPEPOD.
- Not observing the instructions given in the instruction manual provided with the S-CAPEPOD may result in serious physical injury or even death.
- Sharp edges or protrusions may damage the S-CAPEPOD and, therefore, reduce the effectiveness of the product.

CAUTION: Proper use of the S-CAPEPOD requires training and periodic practice. Do not overestimate your skills, strengths and abilities. Staircase evacuations can be difficult. You must be able to control the resident's descent speed while sliding the resident down the staircase. In some instances (heavier residents, a caregiver lacking training or sufficient strength) you should request the assistance of a second caregiver. Periodic practice using the S-CAPEPOD should be an element of evacuation exercises. In all instances, compliance with the instructions is important.

Only use in case of emergency situations. The manufacturer and supplier cannot be held liable for any improper use!

- Practice using the S-CAPEPOD at least once a year in order to guarantee its correct and safe use in the event of an emergency.
- Contact the various rescue services (fire brigade, ambulance service, etc.) to inform them of the purchase and presence of the S-CAPEPOD.
- The standard S-CAPEPOD model is only suitable for beds with a maximum width of 90 cm and a maximum length of 220 cm.
- The S-CAPEPOD is not suitable to be pulled over a naked flame.
- The S-CAPEPOD must be carefully checked for damage after it has been used. If damage is observed, the S-CAPEPOD must not be reused. Furthermore, the S-CAPEPOD must be folded up and put away each time after use.
- Make sure the straps at both the foot and the head of the bed are securely tucked away if the S-CAPEPOD is placed between the bed frame and the mattress and is not used for an emergency evacuation.
- For hygiene reasons, the S-CAPEPOD must be regularly cleaned. Always wash the S-CAPEPOD separately and follow the washing instructions. Closely observe the washing instructions, otherwise the quality of the hook and loop fasteners may be negatively affected.
- The S-CAPEPOD has a 1-year warranty for material and manufacture faults, which is valid from the date of purchase. The manufacturer may ask you for proof of the purchase date. Therefore, it is recommended to keep the invoice in a safe place. Faults or problems with the use of the S-CAPEPOD as a result of an accident, misuse, incorrect use, alterations, vandalism, incorrect placement, incorrect maintenance or administration, as well as overdue normal and routine maintenance, are not covered by the warranty.
- The manufacturer is not liable for (consequential) damage as a result of the use of the S-CAPEPOD.

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